



ViVo MouseTM

The Vortant Video and Voice Mouse
Head Tracker Installation Guide

Version 1.1

Vortant Technologies, LLC
88 High Country Road
Weaverville, NC 28787

Technical Support

Fax: (828) 237-4141
E-mail: support@vortant.com
Telephone: (828) 645-1026 x111

Getting Started

Overview


The ViVo Mouse (pronounced vee voh) is a device that enables users to navigate their computers without the use of a keyboard or mouse. The ViVo Mouse has customizable settings that make it easy to change from voice commands to dwell clicking and back again as needed. Because the equipment used is attached to the computer and not the user, this device can easily be shared among multiple users on the same computer with minimal adjustments.

Before you can begin using ViVo Mouse you must download equipment and program software. The next few pages will guide you. Soon you'll be accessing your computer using video and voice.

Installing the WebCam

This section uses the example of the Logitech Communicate STX or similar webcams. Each model of webcam may have a slightly different installation procedure. In any case, you will need to install your webcam prior to using the ViVo Mouse.

Install the WebCam software CD before plugging in the camera. Do **not** plug in the camera until prompted. If you are using Logitech's QuickCam a series of screens will appear to guide you.

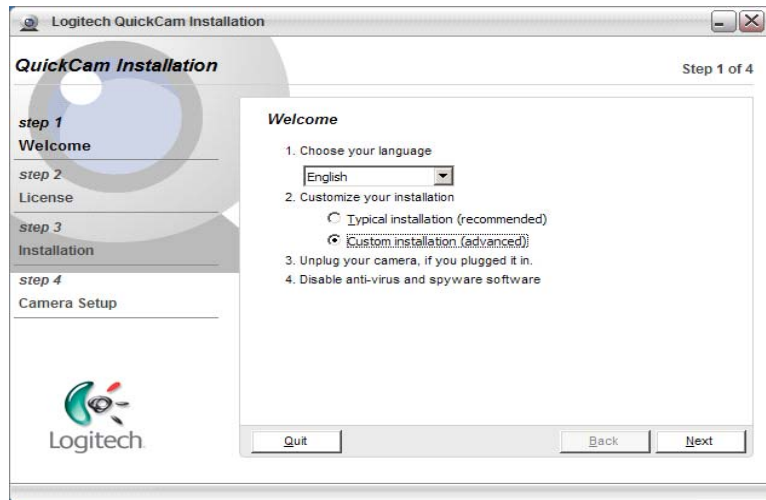


In Step 1, choose your language.

Select the option for Custom installation (advanced)


Follow instructions 3 and 4 before clicking on 'Next.'

On this next screen please select the 2nd option and install only the Logitech QuickCam drivers. Configure the QuickCam to be located in C:\Program Files\Logitech\QuickCam as shown. Then click 'Next.'

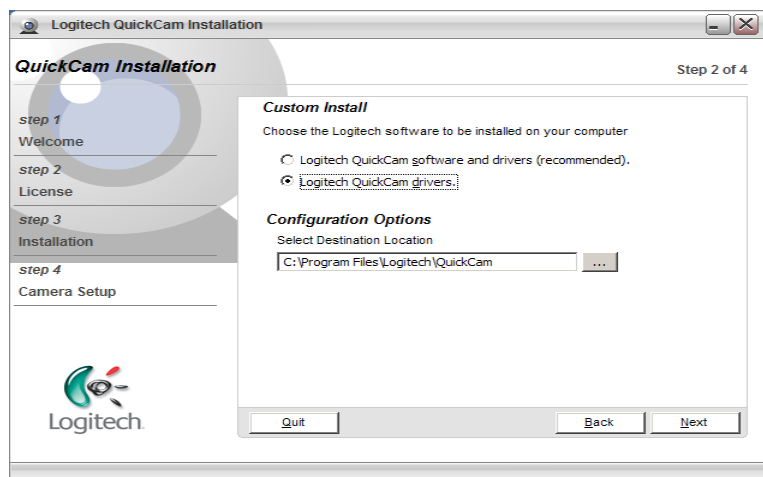


This brings you to the License Agreement. This is standard for most software programs.


Please indicate that you accept the terms of the agreement before clicking 'Next.'

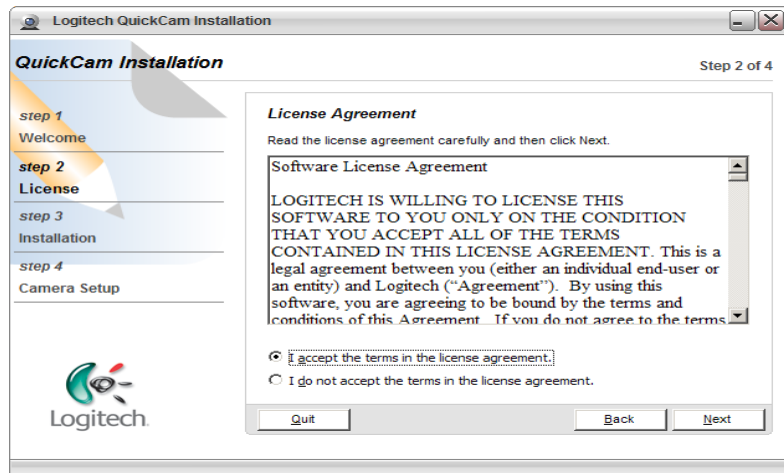

Step 3 shows you the software being downloaded. This may take a moment. Once installation is complete, the 'Next' box will appear darker and you can click on it.

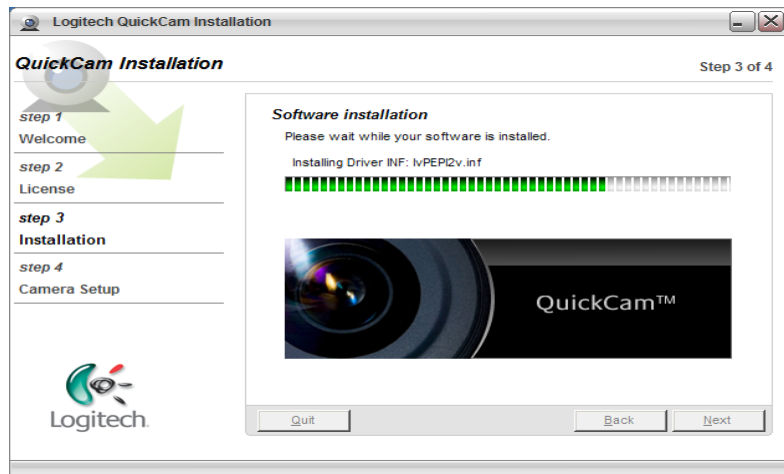


Congratulations! Step 4 indicates your installation has been successful. Click on 'Done.'

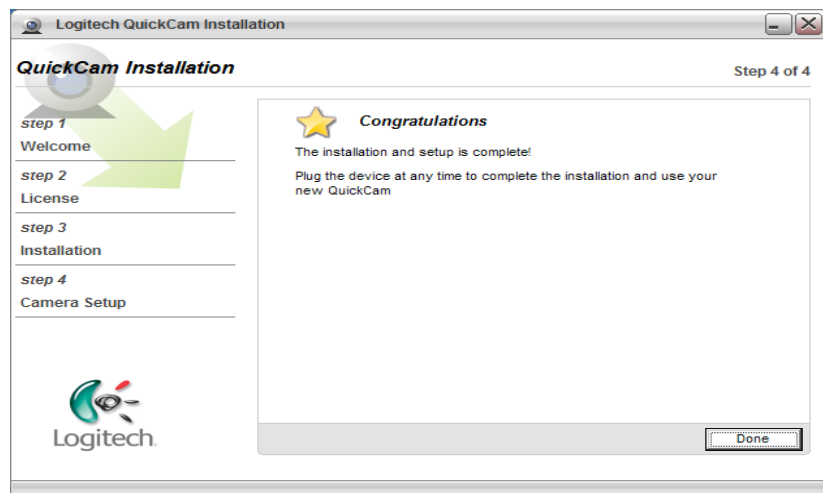

Installing the ViVo Mouse Software

Insert the ViVo Mouse CD into your computer. Click on “Setup” or “Setup.exe” to install the program. Or, if you have received the setup files electronically, Click on “Setup” to install the program.


The setup program will direct you through several steps. Click on “Next” when you see this window.

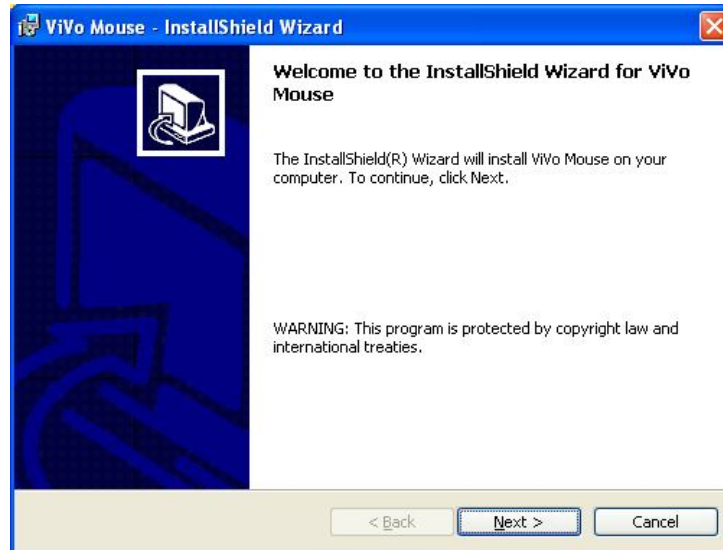


Carefully read the Software License Agreement and if you agree, click on “I accept...” and then on Next.




Enter your name and organization, if any, into the following window:

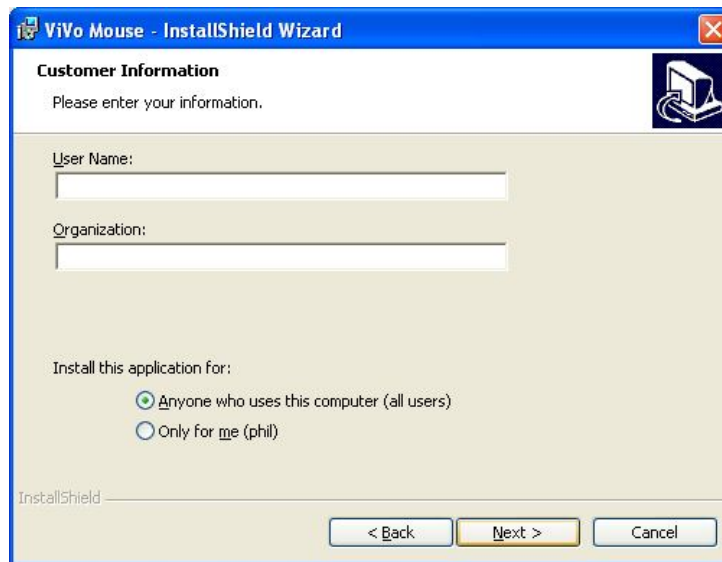


You will see this window when ViVo Mouse is ready to be installed.

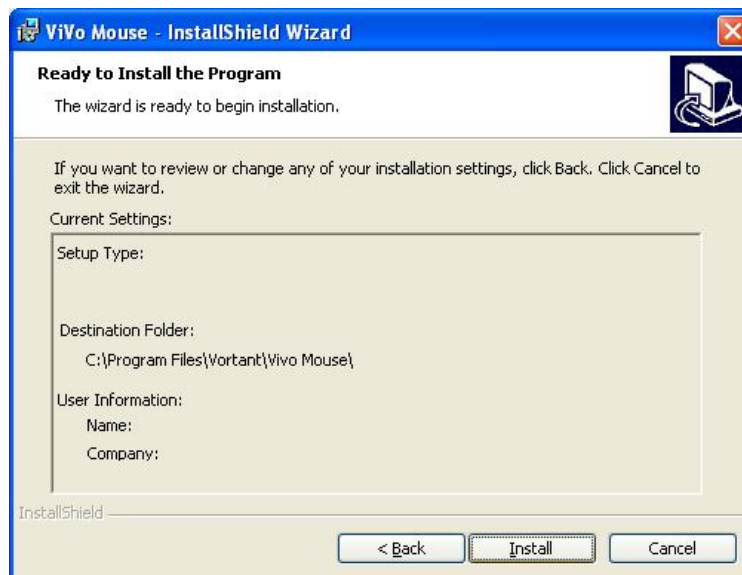

While it is installing, you should see a window like the one below:

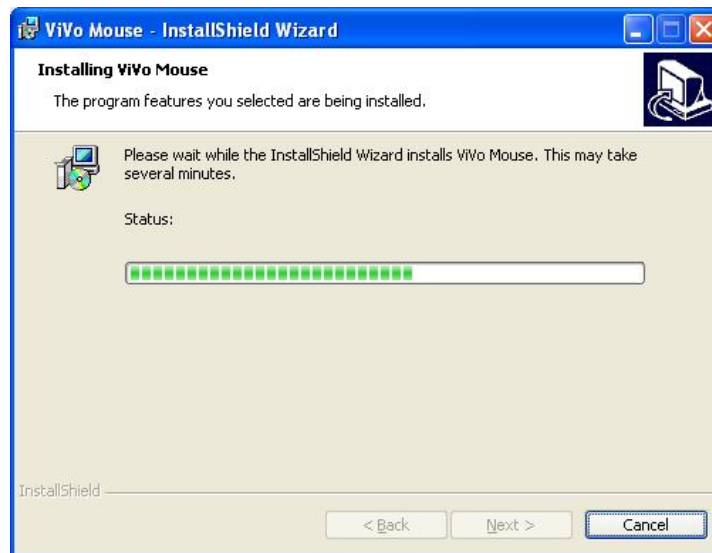

After completion, you may look at the Readme file, which will update you with any important installation information that may be helpful. Be SURE to read this if you are using a Windows Vista computer.

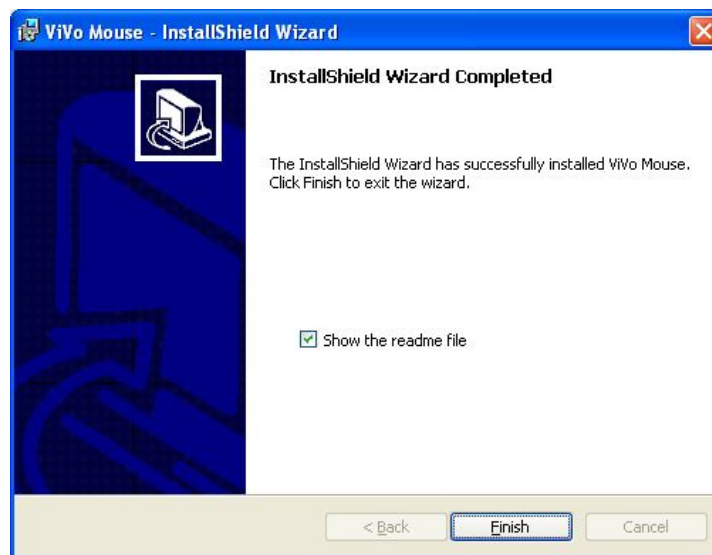


After it is installed, there should be a ViVo Mouse icon that looks something like this on your desktop:



Double click on this icon to start the program. Alternately, go to the Start button and find the ViVo Mouse program on your start menu, probably under the "Vortant" folder.


Activating the ViVo Mouse

The first time you start the ViVo Mouse, you should activate it. Once activated, you will no longer see the following window upon starting the program:

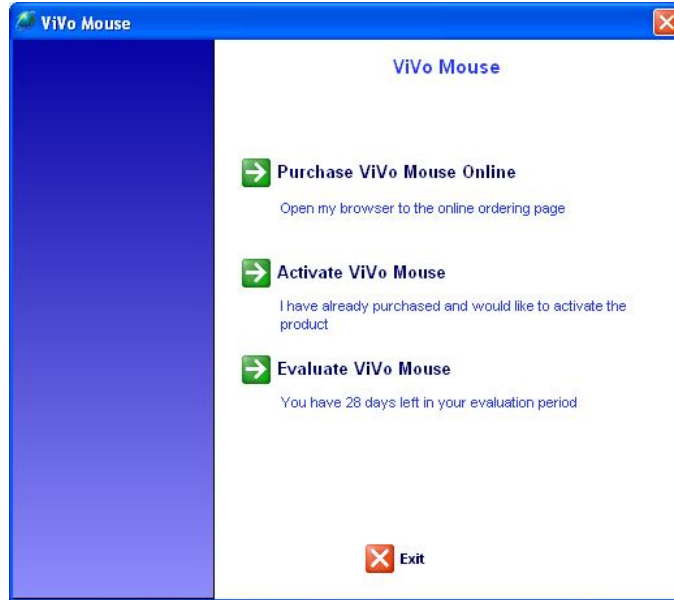


Since you have already purchased the ViVo Mouse, click on the second option “Activate ViVo Mouse”. Then, the following screen should apply.

By far, the easiest and fastest way to activate the ViVo Mouse, if your computer is connected to the Internet, is by Online Activation. Be sure your computer is connected to the Internet, then select that option.

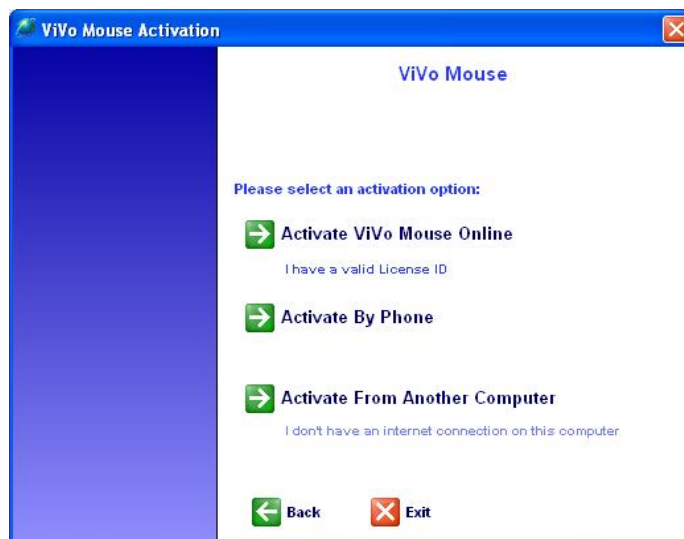

When the following screen appears, find the Activation Codes that you received with your CD-ROM or electronic download of the ViVo Mouse. Enter the License ID (typically a number) into the top box,

and enter the Password (typically letters and numbers) into the bottom box.

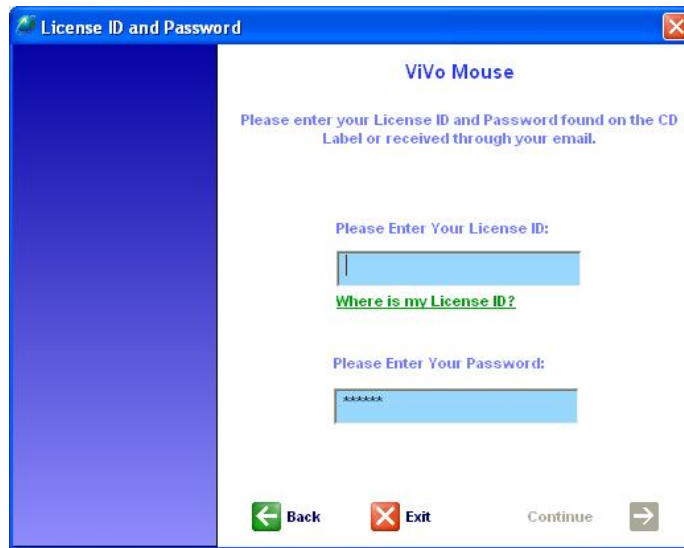

The system will tell you when activation is successful. If it fails the first time, please try carefully typing in the letters and numbers again – remember that it is case-sensitive – capitals and small letters must be typed in exactly as they appear on your password!

If you have trouble, please contact the technical support line as shown in the first page of this Guide.